

TEC-600 Assembly Instructions



Step 1: Remove the Sphygmomanometer from the old cuff assembly.



Step 2: Place the Sphygmomanometer with the TEC-600 parts.




Step 3: Take the single piece of purple tubing and slide it on to "one" side of the retaining posts at the rear of the Sphygmomanometer as shown.



Step 4: Turn the Sphygmomanometer over (Dial facing you) and insert the barbed port from the Sphygmomanometer into the open port of the pump bulb assembly as shown.



Step 5: Turn the Sphygmomanometer over (Rear facing you). Take the free end of the single piece of purple tubing from Step 3 and slide it on to the free retaining post. The dual lumen purple tubing must be underneath the single piece of purple tubing as shown.

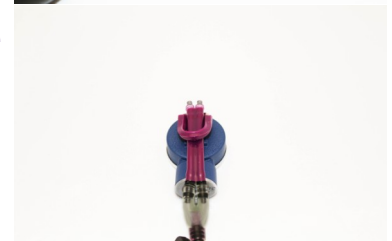

Congratulations! You have now converted your antiquated Sphygmomanometer to a functional handheld unit for use with Technicuff Eco-Friendly Blood Pressure Cuff transducers. Now, simply insert the tubes from the Technicuff Eco-Friendly Blood Pressure Cuff transducer into the clear barbed connectors on the top of the Sphygmomanometer as shown.

